

Tailor made application.

User friendly wound treatment.

Flaminal® is easy to apply for both the patient & health care professional making it ideal for self/shared care.

STEP 1: CHOOSE

- Clean the wound following local protocols and choose which Flaminal® to start with according to exudate levels.



Use **Flaminal® Hydro**
on slightly to moderately
exuding wounds

Use **Flaminal® Forte**
on moderately to highly
exuding wounds



STEP 2: APPLY

- Apply Flaminal® covering the entire wound bed with a sufficiently thick layer (4-5mm). Flaminal® can be applied in a number of ways:



Directly from
the tube



Directly onto
the dressing



With a syringe



With a nozzle




With a spatula

STEP 3: COVER

- Cover Flaminal® with the appropriate secondary dressing according to exudate levels.

Slight Exudate		Transparent film (polyurethane) or non-adherent dressing
Moderate Exudate		Transparent film (polyurethane) or non-adherent dressing or absorbent non-adherent dressing
High Exudate		Absorbent non-adherent dressing



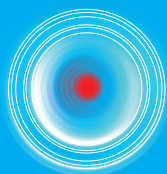

For a variety of wound types.

Due to its unique composition.

Flaminal® for acute and chronic wounds or wounds at risk of infection.

INDICATIONS

- Leg ulcers
- Diabetic foot ulcers
- Superficial & deep partial thickness burns
- Post-surgical wounds
- Traumatic wounds
- Pressure ulcers
- Skin tears
- Wounds from radiotherapy
- Oncology wounds



Infected wounds



Granulating wounds



Sloughy wounds



Necrotic wounds

Flaminal® can be used on infected wounds under medical supervision only

GOOD TO KNOW

If dry whitish alginate flakes appear in the wound:

- when using Flaminal® Forte, the wound is too dry. Step down to Flaminal® Hydro.
- when using Flaminal® Hydro, then ensure the wound is covered with a dressing that retains moisture. The flakes will disappear when the wound moisture balance is restored.

SHELF LIFE

Tubes: up to 12 months after first opening. **Tubs:** up to 1 week after first opening.

ORDERING INFORMATION

Flaminal® Forte



Pack Size	PIP Code	NHS Code
5 x 15g	324-2963	ELG022
1 x 50g	344-9592	ELG023
500g tub	-	ELG028

Flaminal® Hydro



Pack Size	PIP Code	NHS Code
5 x 15g	324-2971	ELG021
1 x 50g	344-9600	ELG025
500g tub	-	ELG029

email info@flenhealth or visit flenhealth.co.uk to get in touch with your local representative

