

# Tailor made application.

User friendly wound treatment.



*Flaminal® is easy to apply for both the patient & health care professional making it ideal for self/shared care.*

## STEP 1: CHOOSE

- Clean the wound following local protocols and choose which Flaminal® to start with according to exudate levels.



Use **Flaminal® Hydro**  
on slightly to moderately  
exuding wounds

Use **Flaminal® Forte**  
on moderately to highly  
exuding wounds



## STEP 2: APPLY

- Apply Flaminal® covering the entire wound bed with a sufficiently thick layer (4-5mm). Flaminal® can be applied in a number of ways:



**Directly from  
the tube**



**Directly onto  
the dressing**



**With a syringe**



**With a nozzle**




**With a spatula**

## STEP 3: COVER

- Cover Flaminal® with the appropriate secondary dressing according to exudate levels.

Slight Exudate		Transparent film (polyurethane) or non-adherent dressing
Moderate Exudate		Transparent film (polyurethane) or non-adherent dressing or absorbent non-adherent dressing
High Exudate		Absorbent non-adherent dressing



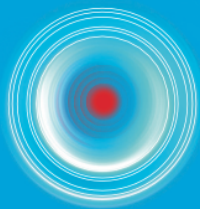

# For a variety of wound types.

Due to its unique composition.

*Flaminal® for acute and chronic wounds or wounds at risk of infection.*

## INDICATIONS

- Leg ulcers
- Diabetic foot ulcers
- Superficial & deep partial thickness burns
- Post-surgical wounds
- Traumatic wounds
- Pressure ulcers
- Skin tears
- Wounds from radiotherapy
- Oncology wounds



**Infected wounds**



**Granulating wounds**



**Sloughy wounds**



**Necrotic wounds**

*Flaminal® can be used on infected wounds under medical supervision only*

## GOOD TO KNOW

If dry whitish alginate flakes appear in the wound:

•when using Flaminal® Forte, the wound is too dry. Step down to Flaminal® Hydro.

•when using Flaminal® Hydro, then ensure the wound is covered with a dressing that retains moisture. The flakes will disappear when the wound moisture balance is restored.

## SHELF LIFE

**Tubes:** up to 12 months after first opening. **Tubs:** up to 1 week after first opening.

## ORDERING INFORMATION

### Flaminal® Forte



Pack Size	PIP Code	NHS Code
5 x 15g	324-2963	ELG022
1 x 50g	344-9592	ELG023
500g tub	-	ELG028

### Flaminal® Hydro



Pack Size	PIP Code	NHS Code
5 x 15g	324-2971	ELG021
1 x 50g	344-9600	ELG025
500g tub	-	ELG029

email [info@flenhealth](mailto:info@flenhealth) or visit [flenhealth.co.uk](http://flenhealth.co.uk) to get in touch with your local representative



*A unique dimension of wound care.*



**Flaminal®**



Doncaster and Bassetlaw  
Teaching Hospitals  
NHS Foundation Trust



Sheffield Teaching Hospitals  
NHS Foundation Trust



Barnsley Hospital  
NHS Foundation Trust



Sheffield Children's  
NHS Foundation Trust



The Rotherham  
NHS Foundation Trust



Rotherham Doncaster  
and South Humber  
NHS Foundation Trust



South West  
Yorkshire Partnership  
NHS Foundation Trust